Bloques 55 DUAL y 55 DEEP, exclusivos de Polidental Zaragoza s.l

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English:

zirconium blocks 55 and 55 DEEP DUAL from Polidental Zaragoza sl

zirconium blocks 55 and 55 DEEP DUAL are uniques from Polidental Zaragoza sl, are a innovation us, to understand it considers that:

- Standards of 55 blocks have a measure of 55 x19 x15, 5 mm (length x width x height) so if you need milling a tooth very long (eg canines, teeth periodontal) ex. despite being a short structure would have to use a larger block, 65x25x22, which rejects much of the zirconium paid, as well as increased wear of water lubricant, burs and milling time.

to troubleshoot and optimize this we developed:

- a new block of 55 (DEEP) to measure 55x15,5x19 (see the differences☺)

55 standard: 55x19x15, 5

55 DEEP: 55x15, 5x19

this simplicity, the user just double the money saved.

DUAL 55 blocks are based on the same concept, but the novelty is block support for 2 positions of use, including 55 standard or 55 DUAL.

to milling: 55 DUAL and 55 DEEP need to update the blocks CEREC library using our free update (Polidental Blocks Update) which can be downloaded from the web:

[www.polidental.com / bloques.html](http://www.polidental.com/bloques.html)

this type of editing in the library of CEREC can be done with our exclusive software Polidental Blocks Editor v07:

functional Windows XP (x32 and x64), Vista (x32 and x64) and Windows 7 (x32 and x64)

- you can create your own completely configurable materials in blocks, with our editor, you can create a block fresable setting:
 - * measures
 - Length x Width x Height

all CEREC users depend on pre-configured blocks of manufacturers, but you can buy the material to be milled directly from manufacturers, hitting the right support and create your own special block, according to their usual practice, and a simple step can clone some pre-established block (keeping the media type) and adapt the measures according to their materials available.

- After you manufacture your new blocks, need take the news measures and create it inside of CEREC software as “milleable block” and will be ready for inLab CEREC milling or Stack.

also with our editor, you can avoid the absurd bar code reading and the waste of time, skip this step saves time = more speed milling.

Why pay for zirconium blocks of a measure that does not fit with its use, and throw a lot of material (zirconium, etc.) to the trash?

it was time to create their own block with CEREC POLDENTAL Block Editor.

is the freedom that you expected.

- For CEREC v3.60 - v3.85

- Works from a flash memory in windows XP, Vista or higher,

download the trial from [www.polidental.com / software.html](http://www.polidental.com/software.html)

is a example of how you can create and customize their own blocks, the blocks created with the trial can not be milled in the CEREC software, an option possible only with full lisensed Polidental Block Editor.

TIP:

You can create your own blocks of zirconia milling remains large blocks, taking off with heat the metal support and paste it on one side, measures this new block, edit it in the CEREC library and milling, approach the 100% of the zirconium blocks payed.