Neu ab:  
New since:  
Nouveau depuis:  
Nuevo desde:  
Nuovo a partire da:


04.2010

**sirona**  
The Dental Company

## CEREC MC XL, inLab MC XL

Ergänzung zur Gebrauchsanweisung, Manuelle Blockspannung verwenden  
Supplement to the Operating Instructions, Using a manual block clamp  
Complément de la notice d'utilisation, Utiliser le serre-bloc manuel  
Anexo al manual del operador, Utilización del sujetabloque manual  
Integrazione alle Istruzioni d'uso, Utilizzo della tensione blocchetto manuale

**Deutsch / English / Français / Español / Italiano**

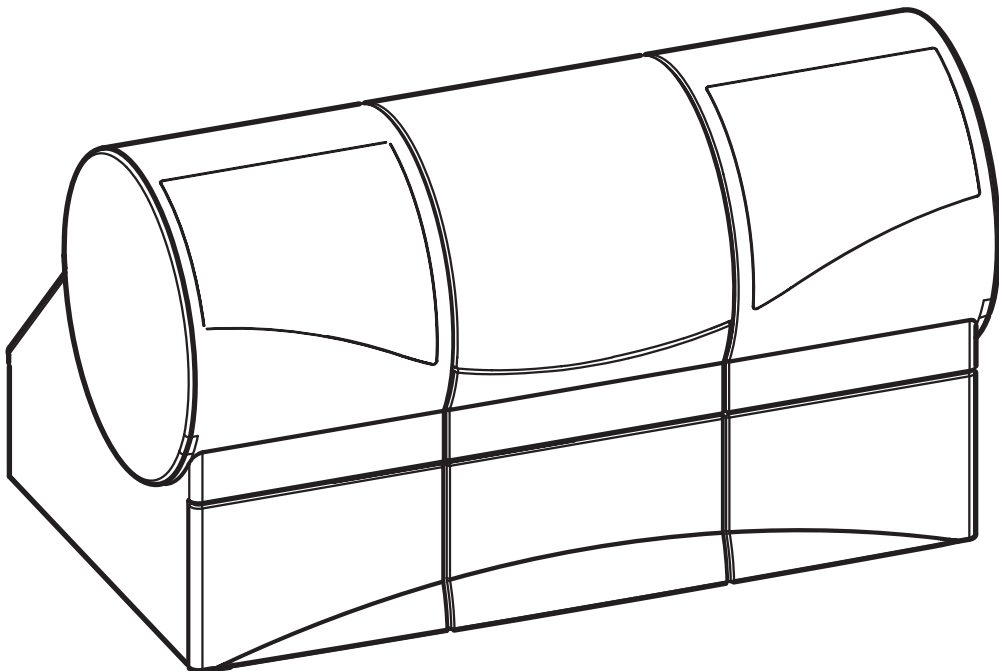Neu ab:

04.2010

# CEREC MC XL, inLab MC XL

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
04.2010

# CEREC MC XL, inLab MC XL

Supplement to Operating Instructions  
Using a manual block clamp

**English**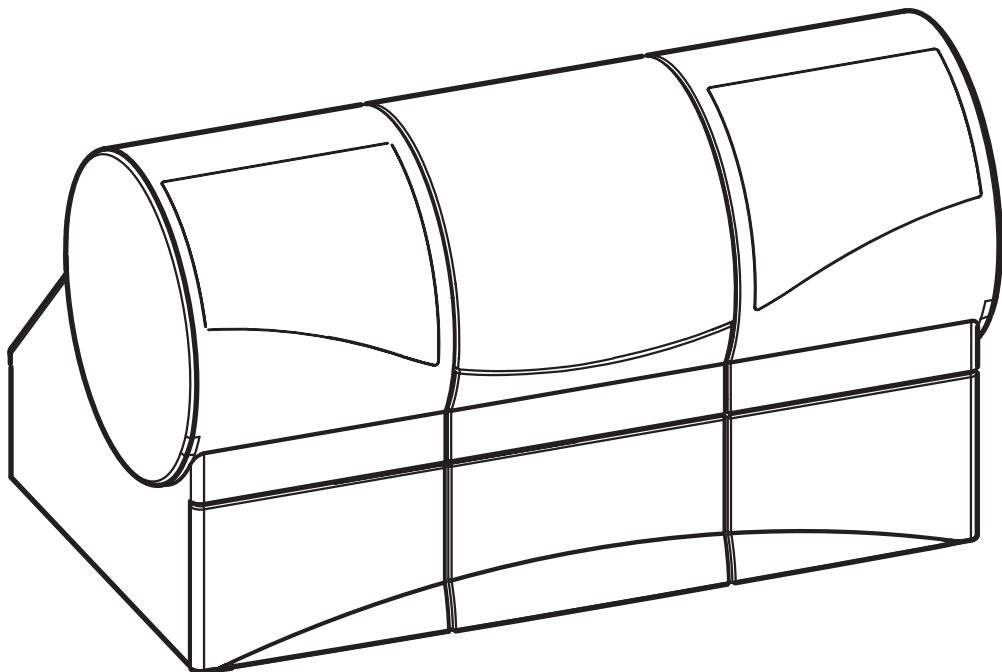

# 1 Introduction

This document describes the use of the manual block clamp. The following chapter "Using a manual block clamp" [ → 3] replaces the procedure described in your Operating Instructions.

File this document in your manual and always proceed as outlined in this description in the future.

## 2 Using the manual block clamp

Store the block clamp tool in the corresponding holder.  
You can attach the holder to a suitable location with the adhesive pad. Clean and degrease the contact surface beforehand.



### NOTICE

#### Wear of the ball pressure screw

The high clamping forces cause wear of the ball pressure screw.

➤ Replace the ball pressure screw every 500 clamping operations.

### Blocks with 10 mm diameter block holders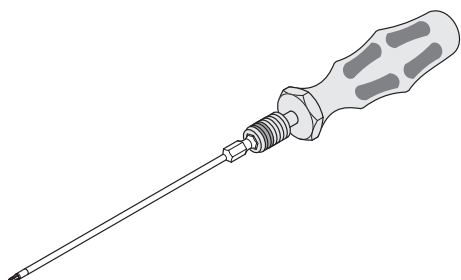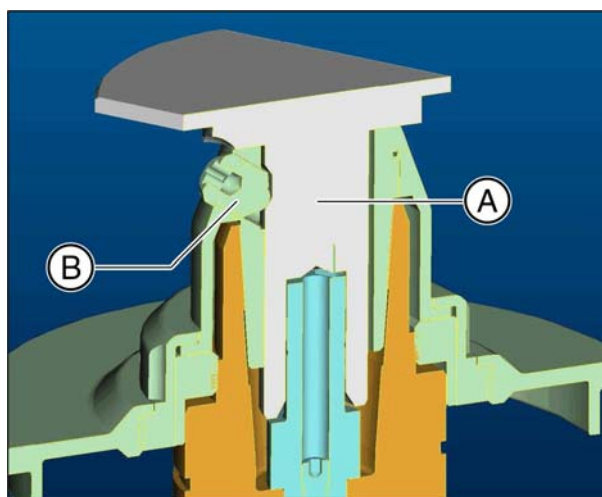



### NOTICE

#### Fasten the block tightly

If the block is not tightened sufficiently, this may result in falsification of the milling result and fracturing of its ceramic material.

- **Fasten the block tightly using the block clamp tool with torque wrench until you hear a click.**
- Check to make sure that the block is seated very firmly.

1. Place the block (A) directly into the block clamp.
2. **Clamp** the ceramic block **securely** with the ball pressure screw (B). Use the block clamp tool with torque wrench for this purpose.
  - ↪ The block is pressed laterally against the contact surface of the block clamp and simultaneously pulled in axially. The plate of the block holder thus rests on the block clamp.

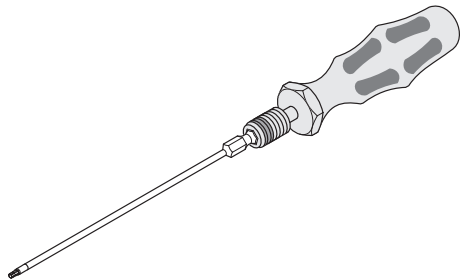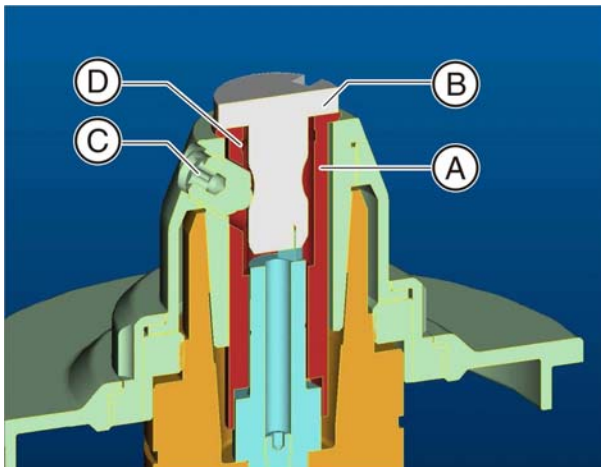**Blocks with 6 mm diameter block holders****NOTICE****Fasten the block tightly**

If the block is not tightened sufficiently, this may result in falsification of the milling result and fracturing of its ceramic material.

- **Fasten the block tightly using the block clamp tool with torque wrench until you hear a click.**
- Check to make sure that the block is seated very firmly.

1. Insert the adapter sleeve (A) into the block clamp.

**NOTICE****Insert the adapter sleeve**

The slot at the bottom end of the adapter sleeve must lie above the radial pin of the block fastener in order to be inserted fully.

The hole for the ball pressure screw is then automatically in the correct position, i.e. coincides with the threaded hole in the block clamp.

2. Insert the block (B) into the block adapter sleeve.
3. **Clamp** the ceramic block **securely** with the ball pressure screw (C). Use the block clamp tool with torque wrench for this purpose.
  - ↳ The block is pressed laterally against the contact surface of the block clamp and simultaneously pulled in axially. The plate of the block holder thus rests on the block clamp.

**Removing the adapter sleeve**

1. Loosen the ball pressure screw.
2. Place the block clamp tool in the inner groove (D) and pull out the adapter sleeve.

### 3 General information

#### NOTICE

After completing a milling process, remove the remainder of the ceramic block out of the block clamp.

#### NOTICE

If you have not used the milling unit for a rather long time, we recommend you should switch it off and then open the milling chamber door so that the milling chamber can dry out.

---

We reserve the right to make any alterations which may be required due to technical improvements.

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© Sirona Dental Systems GmbH 2010
**Sirona Dental Systems GmbH**
Sirona Dental Systems LLC